HYCOOX

Crystal Meso

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2 automated, fast and accurate medical devices
for a truly efficient injection of processing assets.

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Preventive and Restorative Aesthetic

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Optimize each injection procedure.

Modular handpieces and simplified configurable procedures.

HYCOOX and Crystal Meso are adjustable and automated solution dispensing devices that allows significant time saving and better control over the amount of product injected. The needles shape and suction function allows less painful injections and improves patient experience. Ease of use, precise injection depth, the devices allow to succeed its therapeutic and aesthetic procedures safely.

Devices should be handled and used by physicians and healthcare professionals to avoid any risk of harm or complications. Powered by an electric motor, the injectors are adjusted using intuitive digital interfaces. Type of needle used, amount of solution to be injected, speed and number of injections... all parameters of the procedure can be adjusted before treatment. Depending on the purpose of the protocol and the quantity of product to be delivered, the most suitable syringe type can be mounted on the piston holder which will push and inject the solution according to the previously defined settings.

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→ 2 precise and adapted adjustment interfaces to each protocol type.

Hycoox and Crystal Meso offer easy-to-use and intuitive administration interfaces. All options can be adjusted quickly and the user can see the result of the adjustment on the screen. We can adjust parameters such as : syringe size, total number of injections, injection rate, presence or not of aspiration, interval between injection...

→ Varied fields of application with adjustable handpieces.

Hycoox and Crystal Meso injectors have a plunger system that can receive 1CC, 1CC, 3CC, 3CC, 5CC and 10CC syringes with interchangeable heads. This allows many delicate protocols to be executed and optimized after interface setting.

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
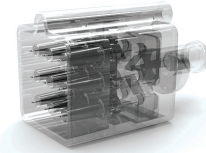
Syringes with specific interchangeable heads :

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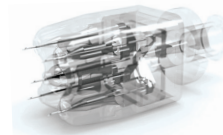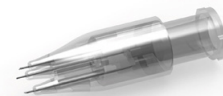
Settings functions :

- 2 injection modes (automatic and manual).
- 6 syringes types 1ccc to 10cc glass and other.
- 4 types of customizable injection procedure.
- 5 injection speeds that adapt to the product viscosity levels.
- 4 suction pressure levels depending on the skin type.
- The interval between each injection of the parameter product.

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1. Syringe holder. 2. Needle head. 3. Setting screen. 4. Suction tube. 5. Settings buttons. 6. Filter. 7. Trigger. 8. Cradle. 9. Spare syringe holders for receiving syringes of different sizes.