English**Description**

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Composition

PBS/Phosphate Buffered Saline), Sodium Hyaluronate (0.7%) Peptide Complex (Oligopeptide-24, Decapeptide-36, Octapeptide-11, Oligopeptide-34, Oligopeptide-92)

Intended purpose

Prostrolane Natural_B is indicated for mid-to-deep dermal implantation for the treatment of damaged subcutaneous tissue caused by chemical, physical and traumatic effects, such as facial wrinkles, folds or nasolabial folds to restore the shape of the face.

Site of Application:

- Face
- Recommended usage volume : 2ml / area (Max usage : 4ml / area)
- Frequency : 1 session / 2 weeks (1 protocol = 2-4 sessions average) (repeat protocol as necessary)

Treatment procedure

Pretreatment Guidelines
Prior to treatment, the patient should avoid taking aspirin, non-steroidal anti-inflammatory medications, St. John's Wort, or high doses of Vitamin E supplements.

These agents may increase bruising and bleeding at the treatment site.

1. It is important to counsel the patient and discuss the appropriate indication, risks, benefits and expected responses to the Prostrolane Natural_B treatment. Advise the patient of the necessary precautions before commencing the procedure.

2. Assess the patient's need for appropriate anaesthetic treatment for managing comfort, i.e., topical anaesthetic, local or nerve block.

3. The patient's face should be washed with soap and water and dried with a clean towel.

Cleanse the area to be treated with alcohol or another suitable antiseptic solution.

4. Sterile gloves are recommended while using Prostrolane Natural_B.

5. Before treatment, press rod carefully until a small droplet is visible at the tip.

6. Prostrolane Natural_B is administered using a thin gauge needle (27G x 13mm or 30G x 13mm) that is sterilized. The sterilized needle is inserted at an approximate angle of 30° parallel to the length of the injected site. Prostrolane Natural_B should be injected into the mid to deep dermis. If Prostrolane Natural_B is injected too superficially this may result in visible lumps and/or bluish discoloration.

7. Inject Prostrolane Natural_B applying even pressure on the plunger rod. It is important that the injection is stopped just before the needle is pulled out of the skin to prevent material from leaking out or ending up too superficially in the skin.

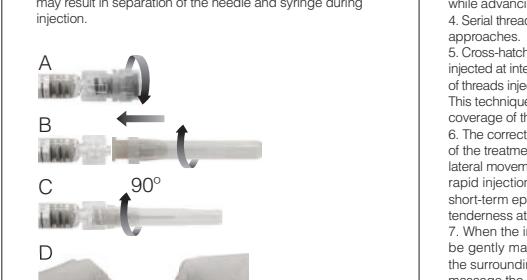
8. Only correct 100% of the desired volume effect. Do not overcorrect. With cutaneous deformities the best result is obtained if the defect can be manually stretched to the point where it is eliminated. The degree and duration of the correction depend on the character of the defect treated.

9. Typical usage for each treatment session is specific to the site as well as wrinkle severity. Or to apply on the cleared skin surface, to wait for full absorption.

10. After treatment, stick the traceability label on the patient's medical record.

Direction for Assembly

Assembly of 27G or 30G needle(Hypodermic needle) to syringe. For safe use of Prostrolane Natural_B, it is important that the needle is properly assembled. Improper assembly may result in separation of the needle and syringe during injection.

(1) Unscrew the tip cap of the syringe carefully(A)
(2) Grasp the narrow part of the needle shield loosely; mount the needle on the Luer-Lock (B) by turning it clockwise until you feel counter-pressure. Grasp the wide part of the needle shield firmly.
(3) Press and turn the needle shield 90° (a quarter turn). The quarter turn is necessary to lock the needle onto the syringe.
(4) Pull off the needle shield.
(5) Use the thumb and forefinger to hold firmly around both the glass syringe barrel and the Luer-Lock adapter. Grasp the needle shield with the other hand. To facilitate proper assembly, both push and rotate firmly.

Contraindications

Prostrolane Natural_B must not be used:
- for the combination with peeling, laser treatment or ultra-sound.
- if the patient has cutaneous disorder, inflammation or infection at the treatment site or near to this site.

Storage condition

Store between 2°C - 25°C. Protect from freezing and sunlight.

Make sure there are no visible signs of damage to the packaging before use.

Shelf life

2 years under recommended conditions.

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